
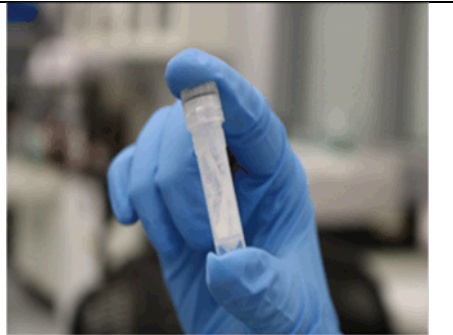
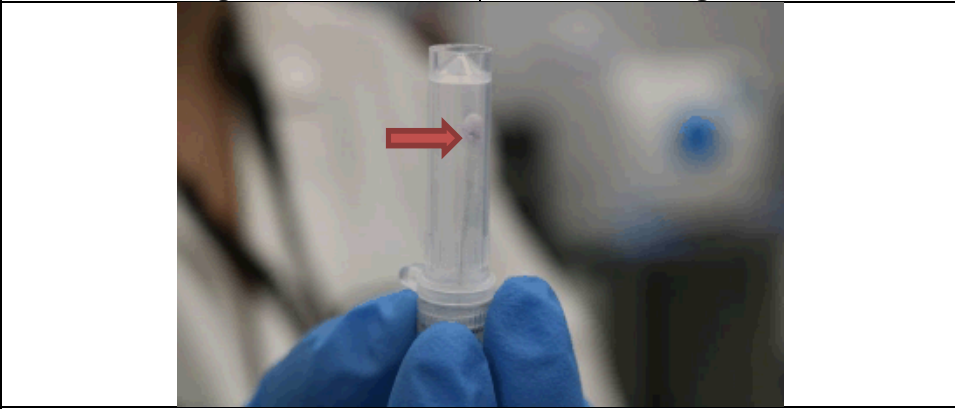


Brand Name of Product	HydroCheck™
Generic Name of Product	HydroCheck™
Product Code Number(s)	HYD-200, HYD-270, HYD-350, HYD-470
Intended Use	To test for residual moisture in cannulated/lumened instruments.
Range of Applications for Product	Detection of residual moisture in cannulated/lumened instruments.
Key Specifications of Product	<ul style="list-style-type: none"> HydroCheck™ test Swab: Available sizes swab sizes: 1.7-, 2.8-, 3.8-, and 5.0 mm. Measuring range: The test kit can detect $\geq 0.05 \mu\text{L}$ of water.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Store in closed box at room temperature (15- to 25 °C (59- to 77 °F) in a dry place.
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To test for residual moisture.
Preparation for Use	<ul style="list-style-type: none"> Put gloves on before using the HydroCheck™ test. Select the correct swab size for the channel or lumen to be tested.
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Figure 1</p> </div> <div style="text-align: center;">  <p>Figure 2</p> </div> </div> <div style="text-align: center; margin-top: 10px;">  <p>Figure 3</p> </div>
Steps for Use of Product	<ul style="list-style-type: none"> Insert the swab into the channel to be tested and feed it all the way through the channel or lumen until it exits. Immediately after the swab exits the channel or lumen, cut the swab approximately 2 cm behind the tip with scissors so the swab tip falls directly into the test vial. (Fig. 1). Close and gently shake at least 3 times. (Fig.2). Reprocess the sampled item after testing with the HydroCheck™.

Interpretation of Test Results	<ul style="list-style-type: none"> • A color change to purple on the swab indicates residual moisture at or above the level of detection. • A small amount of residual moisture on the tip of the swab will be indicated by a small purple spot. (Fig. 3). Large amounts of moisture can turn the whole test purple.
Contraindications of Test Results	<ul style="list-style-type: none"> • Gloves must be worn throughout the test procedure to avoid contamination of the test. • Contact with any source of moisture (e.g. a fingerprint) can give a false-positive result. • If the swab is not tested immediately, it may dry out and give a false-negative result.
Documentation	N/A
Special Warnings and Cautions	Do not open HydroCheck™ test before use.
Disposal	Single use.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	Single use.
Related Healthmark Products	N/A
Other Product Support Documents	N/A
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com