










## Instructions for Use: Decontam Long Sleeve Gloves

<b>Brand Name of Product</b>	Decontam Long Sleeve Gloves
<b>Generic Name of Product</b>	Sleeve Gloves
<b>Product Code Number(s)</b>	41250, 41270, 41280, 41380, 41480, 41580, 41680, 41350, 41370, 41450, 41470, 41570, 41650, 41670, 41270-EX, 41270-EX-ESP, 41280-EX, 41280-EX-ESP, 41370-EX, 41370-EX-ESP, 41470-EX, 41470-EX-ESP, 41570-EX, 41570-EX-ESP, 41670-EX, 41670-EX-ESP
<b>Intended Use</b>	For the handling and protection of contaminated items in the decontamination area of healthcare facilities.
<b>Range of Applications for Product</b>	Reprocessing of clinically used items.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>∅ Fusion of latex free glove and 6 mil vinyl protective sleeve</li> <li>● Comfortable, flexible, and lightweight with elastic cuffing,</li> <li>● Available in various lengths and sizes: <ul style="list-style-type: none"> <li>○ Fifteen (15) mil lined glove code #: 41280, 41380, 41480, 41580, 41680</li> <li>○ Eleven (11) mil lined glove code #: 41270, 41370, 41470, 41570, 41670</li> <li>○ Eleven (11) mil unlined glove code #: 41250, 41350, 41450, 41550, 41650</li> </ul> </li> </ul>

### Shipping & Storage

<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

### Instructions for Using Product

<b>Description of Use(s)</b>	N/A						
<b>Preparation for Use</b>	N/A						
<b>Diagrams (drawings, pictures)</b>	<p style="text-align: center;"><b>DONNING</b></p> <table border="1" style="width: 100%; text-align: center;"> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td><b>1</b></td> <td><b>2</b></td> <td><b>3</b></td> </tr> </table> <p style="text-align: center;"><b>DOFFING</b></p>				<b>1</b>	<b>2</b>	<b>3</b>
							
<b>1</b>	<b>2</b>	<b>3</b>					



<b>Steps for Use of Product</b>	<p><b>Donning:</b></p> <ol style="list-style-type: none"> <li>1. Begin pulling on glove fingers and thumb until they are in the palm of the glove.</li> <li>2. Pull the fingers of the opposite glove so that both hands are into the palm or cuff area of the gloves.</li> <li>3. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand.</li> </ol> <p><b>Doffing (if reused the same day, it is recommended to be cleaned prior to removal and reused by the same person only):</b></p> <ol style="list-style-type: none"> <li>1. Prior to doffing, clean the Sleeve Gloves following the cleaning instructions below in the reprocessing section.</li> <li>2. Begin pulling on glove fingers and thumb until they are in the palm of the glove.</li> <li>3. Pull the fingers of the opposite glove so that both hands are in the palm or cuff area of the gloves.</li> <li>4. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand.</li> <li>5. The sleeves are ready for removal once both hands reach the elastic cuffs.</li> <li>6. Grasp the inside of the opposite sleeve with the thumb and forefinger to remove both sleeve gloves.</li> <li>7. Sleeve Gloves are ready to hang dry or be properly disposed. If cleaning for reuse, do so prior to doffing.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• ⚠ (Prior to donning), carefully inspect the Sleeve Gloves for punctures and cuts.</li> <li>• (If reusing), gloves should only be used by the same person.</li> <li>• Dispose of worn Sleeve Gloves at the end of each workday.</li> </ul> <p>Sleeve gloves are intended to be worn by one-person for one-day.</p> <ul style="list-style-type: none"> <li>• ☠ Treat used Sleeve Gloves as biohazardous.</li> </ul>
<b>Disposal</b>	Treat used gloves as biohazard and dispose of them in accordance with facility policy and procedure for contaminated items.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	Clean with mild neutral pH detergent and rinse with clean water.

	<ul style="list-style-type: none"> <li>● Wash the gloves while they are being worn (using one gloved hand to wash the other).</li> <li>● Rinse the gloves with clean water while being worn.</li> <li>● Doff as instructed above. <ul style="list-style-type: none"> <li>○ If the glove is to be reused, it can only be used by the same person on the same day).</li> <li>○ Hang to dry until next use.</li> </ul> </li> </ul> <p>(Note: Do not remove used Sleeve Gloves from the decontamination area.)</p>
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	PPE Instrument Retrieval Brochure, PPE Instrument Retrieval Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark, A Getinge Company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com