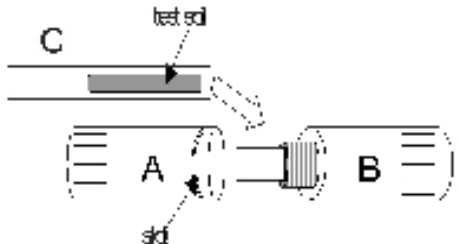


<b>Brand Name of Product</b>	LumCheck™
<b>Generic Name of Product</b>	Lumen cleaning challenge device
<b>Product Code Number(s)</b>	WLC-101, WLC-102
<b>Purpose of Product</b>	Challenge automated lumen cleaning function of cleaning machines.
<b>Range of Applications for Product</b>	May be used in cleaning equipment intended to flush lumened instruments.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Coupon with a test soil comprised of blood proteins (typically found inside of clinically used MIS and other lumened/channeled items).</li> <li>• Single-channel lumen holder designed as a surrogate of the internal channel of an MIS instrument or other lumened items. The holder unscrews for placement of the test coupon.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	Store in the protective foil pouch at a temperature of two (2)- to 25 °C.
<b>Packaging Contents</b>	<ul style="list-style-type: none"> <li>• Each coupon is placed in an individual water-tight foil pouch.</li> <li>• Twenty-five (25) tests per box.</li> <li>• Holder sold separately.</li> </ul>
<b>Shelf Life</b>	Eighteen (18) months from date of manufacture. (Note: Consult package for expiration date.)

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Challenge automated lumen cleaning function of instrument cleaning machines with a standardized soil test inside a surrogate device.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures):</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Unscrew the LumCheck™ holder.</li> <li>2. Detach part A (marked with “slot”) from part B.</li> <li>3. Open the protective pouch of LumCheck™ and insert the test coupon (part C) in part B. Do not touch the area covered with the test soil.</li> <li>4. Close LumCheck™ holder by screwing part A into part B.</li> <li>5. Connect the holder with one of the channel irrigation systems of the washer (luer-lock). (Note: Consider using a minimum of one (1) LumChek™ test per level if utilizing a multi-level ultrasonic cleaner).</li> <li>6. Start the wash cycle according to the manufacturer’s IFU.</li> <li>7. At the end of the cycle, retrieve the holder from the washer.</li> <li>8. Open the holder by unscrewing part A from part B and remove the test coupon without touching the soiled areas.</li> <li>9. Visually inspect the coupon for residual soil. Compare the test result to the <a href="#">interpretation chart</a>, WT-104, scale (0–5).</li> </ol>
<b>Interpretation of Results</b>	<p>If less than optimal results were obtained ("Steps for Use of Products" 1–5), make adjustments to the equipment utilizing the <a href="#">chart (WT-104)</a> as a guide. Typical failures can be caused by:</p> <ul style="list-style-type: none"> <li>• Insufficient detergent due to empty containers.</li> <li>• Detergent pump failure or clogged delivery tubing.</li> <li>• Poorly functioning water pump.</li> <li>• Incorrect water temperature.</li> <li>• Incorrect cycle settings.</li> </ul>

<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	Record results on <a href="#">log sheet</a> located on hmark.com.
<b>Special Warnings and Cautions</b>	N/A
<b>Disposal</b>	Dispose of in a biohazard container.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	TOSI®
<b>Other Product Support Documents</b>	ProFormance™ Brochure, ProFormance™ Price List
<b>Reference Documents</b>	LumCheck™ Videos, LumCheck™ Log Sheet, LumCheck™ Example Policy, Instructions for Ultrasonic & Lumen Tests
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com