

<b>Brand Name of Product</b>	Magnetic Mats
<b>Generic Name of Product</b>	Mats
<b>Product Code Number(s)</b>	11116
<b>Intended Use</b>	Magnetic Mats provide a magnetized non-slip surface that minimizes accidental injuries and droppage by providing an alternative to hand-to-hand transfer.
<b>Range of Applications for Product</b>	Hands-free transfer
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Silicone Rubber, magnetized, non-slip surface</li> <li>• Color: Purple</li> <li>• (L) 12- x (W) 16 inches</li> <li>• Autoclavable</li> <li>• Special cutting lines are placed on the mat for cutting it into two smaller mats as necessary for smaller procedures.</li> <li>• Waterproof.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	Store and handle with care.
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Magnetic Mats provide a magnetized non-slip surface that minimizes accidental injuries and droppage by providing an alternative to hand-to-hand transfer.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	N/A
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Lie the Magnetic Mat flat or secure it through the holes at the corners on an uneven surface.</li> <li>2. Do not perforate any other location on the Magnetic Mat.</li> <li>3. Place item on Magnetic Mat when not in use or when non-hand-to-hand transfer is desired.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Do not place on or near any patients with a pacemaker or similar device without first checking compatibility with such devices.</li> <li>• Avoid contact with the following substances: <ul style="list-style-type: none"> <li>○ Oils, ointment, and oil containing soaps (green soaps)</li> <li>○ <i>Hydrocarbon solvents</i>: ether, acetone and hydrochloric acids, lye, and sodium hypochlorite (bleach)</li> <li>○ Phenol and hexachlorophene products</li> <li>○ Quaternary compounds and benzalkonium chloride.</li> </ul> </li> </ul>
<b>Disposal</b>	This item is to be disposed as ☒ biohazardous waste in compliance with the healthcare facility's policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual:</b>	<ul style="list-style-type: none"> <li>• Wash the Magnetic Mat with mild detergent.</li> <li>• Rinse with water and dry with non-linting material.</li> </ul>
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying:</b>	N/A

<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Magnetic Mats should be sterilized flat without folding and should not be placed on a magnetic metallic surface during steam sterilization.</li> <li>• Recommended cycle time for steam sterilization: <ul style="list-style-type: none"> <li>○ 132 °C (270 °F) for 4 minutes</li> <li>○ 135 °C (275 °F) for 3 minutes.</li> </ul> </li> <li>• Gravity cycle at 121 °C (250 °F) for 30 minutes.</li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	ProTech® Brochure, ProTech® Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com