





<b>Brand Name of Product</b>	SonoCheck™ Hook
<b>Generic Name of Product</b>	SonoCheck™ Hook
<b>Product Code Number(s)</b>	TI108-HK
<b>Intended Use</b>	The hook securely holds down the SonoCheck™ vial to the basket during the ultrasonic cycle.
<b>Range of Applications for Product</b>	Hold the SonoCheck™ vial in the tray.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Hook-shaped design</li> <li>• Stainless steel</li> <li>• Reuseable hook.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To secure the SonoCheck™ vial in place during testing.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	<div style="display: flex; flex-wrap: wrap; justify-content: space-around;"> <div style="text-align: center;">  <p><b>Figure 1</b></p> </div> <div style="text-align: center;">  <p><b>Figure 1b</b></p> </div> <div style="text-align: center;">  <p><b>Figure 2 (Wire Basket)</b></p> </div> <div style="text-align: center;">  <p><b>Figure 2b (Wire Basket)</b></p> </div> </div>

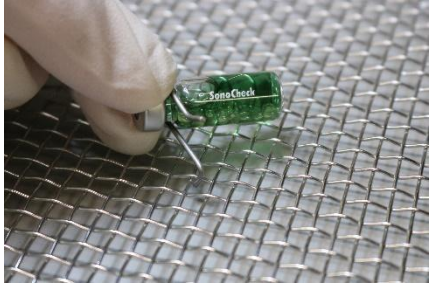

**Figure 2c (Perforated Basket)**



**Figure 2d (Perforated Basket)**



**Figure 3**



**Figure 4a**



**Figure 4b**



**Figure 4c**

**Steps for Use of Product**

**Before the SonoCheck™ test:**

1. Place the hook around the neck of the SonoCheck™ capsule.
2. Press the hook down so that the groove of the hook fits securely around the neck of the capsule. **(Fig. 1 and 1b)**.
3. Insert the L-shaped hook on the left side into the hole of the instrument basket to hold down the capsule. The hook can be attached to a wire basket and perforated basket. **(Fig. 2a–d)**.
4. Place the SonoCheck™ in the correct location with the tray/basket according to SonoCheck™ IFU placement based on the size of the tank.

**After the SonoCheck™ test is complete:**

5. Hold the glass capsule with three fingers of one hand and the index finger and thumb of the other hand on the hook and lid of capsule. **(Fig. 3)**.
6. Place the index finger on the L shaped hook, then place the thumb on the lid of the capsule for support **(Fig. 4a)**. Push the hook back off the neck of the capsule **(Fig. 4b)** and remove the hook with both fingers **(Fig. 4c)**.

**Contraindications of Test Results**

N/A

**Documentation**

N/A

**Special Warnings and Cautions**

N/A

**Disposal**

N/A

**Reprocessing Instructions**

**Point of Use**

N/A

**Preparation for Decontamination**

N/A

**Disassembly Instructions**

N/A

**Cleaning – Manual**

N/A

N/A

<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	Dry after use and store in dry place.
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	N/A
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com