



Instructions for Use: Cool Aids Headwear

Brand Name of Product	Cool Aids
Generic Name of Product	Headwear
Product Code Number(s)	ECB-001, ECB-002, ECB-003, ECB-004, ECS-001, ECS-002, ECS-003, ECBC-WT
Purpose of Product	To provide a method of keeping the employee's scalp/head cool and comfortable, while working in the decontamination areas.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> • Cool Aids Skull Cap: Comfortable, lightweight, and durable. <ul style="list-style-type: none"> ○ Outer fabric is made of 100% polyester mesh ○ Inner fabric is 53% Polymer and 47% Cellulose. The tie fastener in the back ensures a comfortable fit. • Cool Aids Beanie: Same fabric content as the Cool Aids Skull Cap. Rear elastic panel provides a comfortable fit. • Cooling Sports Cap: Mesh polyester outer with Cool Aids inner fabric and water repellent nylon liner. • Custom Imprinted Cooling Beanie: Same fabric content as the Cool Aids Skull Cap. Rear elastic panel provides a comfortable fit. • Provides three (3) to five (5) hours of cool relief per soaking.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	N/A
Preparation for Use	<ol style="list-style-type: none"> 1. Submerge skull cap/beanie in cool water for one (1) to two (2) minutes. 2. Remove and gently squeeze dry. <p>Skull cap/beanie will remain cool for a few hours. Reactivate by submerging in water, as needed.</p> <p>NOTE: Complete one wet/dry cycle before using.</p>
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ul style="list-style-type: none"> • Cool Aids Skull Cap <ol style="list-style-type: none"> 1. Apply comfortably over the scalp. 2. Tie the back to fasten. 3. Surgical headwear fits easily over the Cool Aids Skull Cap. • Cool Aids Beanie <ol style="list-style-type: none"> 1. Apply comfortably over the scalp with the elastic panel at the back. 2. Surgical headwear fits easily over the Cool Aids Beanie. • Cool Aids Sports Cap <ol style="list-style-type: none"> 1. Apply comfortably to the head. 2. Hook and loop size adjustment on back of the hat. • Custom Imprinted Cooling Beanie <ol style="list-style-type: none"> 1. Apply comfortably over the scalp with the elastic panel at the back. 2. Surgical headwear fits easily over the Cool Aids Beanie.
Interpretation of Results	N/A
Contraindications of Test Results	N/A

Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<ol style="list-style-type: none"> 1. Handwash in warm water using a mild soap. 2. Rinse and hang to dry. <p>NOTE: Cleaning/washing of these products should be done in a clean area. Item should then be hung up in a clean area to dry. Keep in a clean area between uses.</p>
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com