

<b>Brand Name of Product</b>	Flex Arm
<b>Generic Name of Product</b>	Flexible arm mount for Flexible Inspection Scope
<b>Product Code Number(s)</b>	100657-K
<b>Intended Use</b>	To hold the Flexible Inspection Scope in place while visually inspecting for residual soil inside cannulated items.
<b>Range of Applications for Product</b>	Healthmark's Flexible Inspection Scope.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Flexible steel arm</li> <li>• Base fitting C-Clamp</li> <li>• Matte black vinyl</li> <li>• 2 foot long</li> <li>• Holder for Flexible Inspection Scope</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Contents</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	




<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	To hold the Flexible Inspection Scope in place to visually inspect for remaining soils inside of cannulated items.
<b>Preparation for Use</b>	<ul style="list-style-type: none"> <li>• Remove Flex Arm from packaging.</li> <li>• Straighten the Flex Arm so it stands vertically.</li> </ul>
<b>Diagrams (drawings, pictures)</b>	   <p style="text-align: center;"> <b>Figure 1</b>                      <b>Figure 2</b>                      <b>Figure 3</b> </p>



Figure 4



Figure 5



Figure 6



**Figure 7**



**Figure 8**



**Figure 9**



**Figure 10**

**Steps for Use of Product**

1. Loosen the vice at the bottom of the Flex Arm. **Fig.1**
2. Secure the arm to table by adjusting the vice until the base is locked in position and stable. **Fig. 2**
3. Make sure the microphone holder is secured into place. The black button on the side of the neck should be clicked into place. **Fig. 3**
4. To remove the neck of the microphone holder, press the black button on the side of the neck that holds the microphone holder then lift the neck and the microphone holder up. **Fig. 4**
5. To reattach the neck, press the black button inward and slide the neck of the microphone holder down into the groove of the Flex Arm. Make sure the button clicks back into place (lift the neck up to ensure it is securely locked in).
6. To detach the neck of the microphone holder from the Flex Arm, grip the neck and loosen the silver bolt and neck counterclockwise, lift up from the Flex Arm. **Fig. 5**
7. Place the flexible working length through the microphone holder and place the handpiece inside of the holder. **Fig. 6**
8. Make sure the handpiece is secured inside the microphone holder with the buttons facing upwards. **Fig. 7**
9. Adjust the Flex Arm into the desired position. **Fig. 8**  
Next, tilt the microphone holder up or down to the desired position for the FIS-005. **Fig. 9**
10. To remove the FIS-005 from the microphone holder, grip the back of the FIS-005 handpiece then pull backwards to release from the holder until the flexible working length is completely removed from the holder. **Fig. 10**

<b>Interpretation of Test Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>• Use with Isopropyl Alcohol wipes per the wipe manufacturer’s IFU.</li> </ul>
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>