

Instructions for Use: Nic-Nac Washer Baskets & Balls

Brand Name of Product	Nic-Nac Washer Baskets & Balls
Generic Name of Product	Washer baskets for small items
Product Code Number(s)	NNB001, 21000-H, NNB002, NNB003, NIC000, NIC MICRO 1, NIC MICRO 2A, NIC MICRO 2, NIC MICRO 3, CIL-004, CIL-005.
Purpose of Product	To hold small items when washing inside an automated instrument and/or ultrasonic washer (preventing damage and loss).
Range of Applications for Product	Small items that could be blown around, damaged, or lost in an automated instrument washer and ultrasonic washer.
Key Specifications of Product	<ul style="list-style-type: none"> ● Manufactured from stainless steel. ● Hinged lid for safe containment of small parts. ● Mesh design allows thorough exposure to cleaning and rinsing solution(s) within an automated instrument washer and ultrasonic.

Shipping & Storage

Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product

Description of Use(s)	Place small items in the washer basket that often get lost or damaged inside of an automated instrument washer or ultrasonic.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place small items in basket. 2. Close lid of basket. 3. Place basket into instrument washer for cleaning.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions

Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> ● Compatible with steam sterilization. ● Recommended cycle: <ul style="list-style-type: none"> ○ 132 °C (270 °F) for 4 minutes. ○ 135 °C (275 °F) for 3 minutes in a pre-vac sterilizer. ● Maximum temperature is 135 °C (275 °F) for 30 minutes.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Washer Baskets
------------------------------------	----------------

Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com