

Brand Name of Product	Clean Test Swabs
Generic Name of Product	Swabs for extraction of residuals for testing
Product Code Number(s)	SWTD-110-12, SWTD-200-12, SWTD-270-12, SWTD-350-12, SWTD-420-12, SWTD-470-12
Purpose of Product	For use in combination with ProChek®-II or HemoCheck™ to detect residual protein or hemoglobin.
Range of Applications for Product	Detection of hemoglobin or protein residues on surfaces. These swabs are for use with conjunction with the ProChek®-II or HemoCheck™.
Key Specifications of Product	

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	These swabs are for use with conjunction with the ProChek®-II or HemoCheck™ to detect residual protein or hemoglobin.
Preparation for Use	Select the swab based upon the diameter and length of the channel to be tested.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Swab the target area. <ol style="list-style-type: none"> a. Wet surfaces are swabbed with the dry swab. b. Dry surfaces are swabbed by moistening the swab with a drop of clean water. (NOTE: Do not use chlorinated water.) 2. Be sure to extend the swab beyond the end of the item. 3. Cut the stem with supplied scissors into the reagent vial. (NOTE: see IFU for the relevant reagent test.) 4. Depending on the reagent test being used, follow the ProChek®-II or HemoCheck™ IFU for testing instructions.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Since it is possible that contaminants are present, it is recommended to dispose of the Clean Test Swabs in a ☒ biohazard container.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	ProChek® -II or HemoCheck™
------------------------------------	----------------------------

Other Product Support Documents	N/A
Reference Documents	Improving the Quality of Surgical Trays, Kovach; Loaner Instrumentation – Processing the Unknown, AORN Journal; Study on HemoCheck™, Pfiefer
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com