













Instructions for Use: Microdenier Wipes

Brand Name of Product	Ultra Microdenier Wipes
Generic Name of Product	Ultra Microdenier Cleaning Wipes
Product Code Number(s)	TX59
Purpose of Product	For cleaning surfaces requiring low level disinfection.
Range of Applications for Product	The Ultra Microdenier Wipes were specifically developed with an ability to perform over a wide range of applications and settings. Ideal for surfaces that are susceptible to scratching and items that are susceptible to damage by liquid or aerosol cleaners and disinfectants.
Key Specifications of Product	<ul style="list-style-type: none"> • Ultra Microdenier yarn • Capable of removing microbial and other contamination from surfaces without use of liquid solvents • Very tight, stable fabric construction • 100% polyester continuous filament yarn

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product							
Description of Use (s)	To allow for safe cleaning of sensitive instruments and fixtures, as well as all general surfaces.						
Preparation for Use	1. Be sure to consult and follow closely the device manufacturer's IFU.						
Diagrams (drawings, pictures)	<table border="1"> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td>Fig. 1A unfolded wipe</td> <td>Fig. 1B 1/2 folded wipe</td> <td>Fig. 1C 1/4 folded wipe</td> </tr> </table>  <p>Fig.2</p>				Fig. 1A unfolded wipe	Fig. 1B 1/2 folded wipe	Fig. 1C 1/4 folded wipe
							
Fig. 1A unfolded wipe	Fig. 1B 1/2 folded wipe	Fig. 1C 1/4 folded wipe					
Steps for Use of Product	<ol style="list-style-type: none"> 2. Follow relevant site protocol (procedures for safety, contamination, etc.) and wear appropriate PPE according to facility policies. 3. Fold wiper in mid-air into quarter folds (Fig. 1A–1C). This will produce several clean surface areas and allow better contact with the surface to be wiped. 4. When wiping, hold the wiper so that the folded edge is toward the area to be wiped. Hold the unfolded edges in your hand. Group the unfolded edges between thumb and forefinger. Wipe in one direction, overlapping wiped area by 10% to 25%. 5. Wipe from cleanest to least clean regions of the surface being wiped. Wipe systematically, for example, from top to bottom, far to near. (Fig. 2) 6. Keep track of which surfaces have been cleaned and which wiper areas are 						

	<p>unused.</p> <p>7. Always use the cleanest surface of the wiper. If re-wiping use a clean portion of the wiper, not the used wiper area.</p>
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	Be sure to consult the IFU of the device manufacturer to be sure full compliance with their instructions.
Disposal	Treat as biohazard and discard of in compliance with facility guidelines.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	DuraSponge™, Lint Free Wipes
Other Product Support Documents	Prosys Price Brochure, Prosys Price List
Reference Documents	
Customer Service Contact	<p>Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com</p>