

|  |  |
|--|--|
| <b>Brand Name of Product</b>             | Luer Lock Pre-filled Syringe & Slip Tip Adapter  |
| <b>Generic Name of Product</b>           | Luer Lock Syringe & Slip Tip Adapter   |
| <b>Product Code Number(s)</b>            | SW-010, SLTP-01  |
| <b>Intended Use</b>                      | Designed to be used to extract a sample for cleaning verification testing.   |
| <b>Range of Applications for Product</b> | Use to flush lumened instruments.  |
| <b>Key Specifications of Product</b>     | <p>Luer Lock Pre-filled Syringe: SW-010</p> <ul style="list-style-type: none"> <li>• Clear</li> <li>• 10 mL Water Luer Lock Pre-filled Syringe.</li> </ul> <p><i>Optional (sold separately):</i><br/>Slip Tip Adapter: SLTP-01</p> <ul style="list-style-type: none"> <li>• Beige</li> <li>• 1.25- Length x .063 (63/1000)-inches Diameter</li> <li>• High-density polypropylene.</li> </ul> |

|   |   |
|---|---|
| <b>Shipping &amp; Storage</b>                 |   |
| <b>Shipping Conditions &amp; Requirements</b> | N/A                                       |
| <b>Storage Conditions</b>                     | N/A                                       |
| <b>Packaging Contents</b>                     | SW-011: Box of 100<br>SLTP-01: Box of 100 |
| <b>Shelf Life</b>                             | N/A                                       |

|  |   |
|--|---|
| <b>Instructions for Using Product</b>    |   |
| <b>Description of Use(s)</b>             | Use to extract samples for cleaning verification testing.   |
| <b>Preparation for Use</b>               | N/A   |
| <b>Diagrams (drawings, pictures)</b>     | N/A   |
| <b>Steps for Use of Product</b>          | <ol style="list-style-type: none"> <li>1. Remove the white cap from the tip of the 10 mL Luer Lock Pre-filled Syringe.</li> <li>2. <b>Optional – Sold Separately:</b> Place the Slip Tip Adapter onto the tip of the syringe and twist to secure the Slip Tip Adapter.</li> <li>3. Insert the 10 mL Luer Lock Pre-filled Syringe tip into the channel port.</li> <li>4. Press down on the plunger rod with your thumb and the plunger tip will push the water into the channel port.</li> </ol> |
| <b>Interpretation of Test Results</b>    | N/A   |
| <b>Contraindications of Test Results</b> | N/A   |
| <b>Documentation</b>                     | N/A   |
| <b>Special Warnings and Cautions</b>     | N/A   |
| <b>Disposal</b>                          | <ul style="list-style-type: none"> <li>• Single use.</li> <li>• Dispose of in accordance with facility policy.</li> </ul>   |

|   |     |
|---|-----|
| <b>Reprocessing Instructions</b>            |     |
| <b>Point of Use</b>                         | N/A |
| <b>Preparation for Decontamination</b>      | N/A |
| <b>Disassembly Instructions</b>             | N/A |
| <b>Cleaning – Manual</b>                    | N/A |
| <b>Cleaning – Automated</b>                 | N/A |
| <b>Disinfection</b>                         | N/A |
| <b>Drying</b>                               | N/A |
| <b>Maintenance, Inspection, and Testing</b> | N/A |
| <b>Reassembly Instructions</b>              | N/A |
| <b>Packaging</b>                            | N/A |
| <b>Sterilization</b>                        | N/A |
| <b>Storage</b>                              | N/A |

|  |   |
|--|---|
| <b>Additional Information</b>          | N/A   |
| <b>Related Healthmark Products</b>     | ChannelCheck, ChannelCheck Convenience Pack   |
| <b>Other Product Support Documents</b> | N/A   |
| <b>Reference Documents</b>             | N/A   |
| <b>Customer Service Contact</b>        | Healthmark, A Getinge company<br>18600 Malyn Blvd.<br>Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br>hmark.com |