

Brand Name of Product	ScopeDry Check™
Generic Name of Product	ScopeDryCheck™
Product Code Number(s)	SDC-001
Intended Use	To test lumens/channels for detectable levels of residual moisture.
Range of Applications for Product	For any lumened/channeled items that may contain residual moisture.
Key Specifications of Product	<ul style="list-style-type: none"> Measuring range: Detection down to 50 nL (or 0.05 µL) of residual moisture. ScopeDryCheck™ Card: 3.5- (W) x 2 inches (H). Response time: 2 to 5 seconds.

Shipping & Storage	
Shipping Conditions & Requirements	Avoid exposure to moisture.
Storage Conditions	<ul style="list-style-type: none"> Store in a cool dry place at 60- to 80 °F with 30–60% rh. Avoid exposure to moisture. Avoid exposure to UV light.
Packaging Contents	N/A
Shelf Life	Two (2) years from the date of manufacture (Mfr.).

Instructions for Using Product	
Description of Use(s)	To test lumens/channels for detectable levels of residual moisture.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> With dry gloved hands, after drying any lumened/channeled items according to the Mfr.'s IFU, with the same air source used to dry the instrument, flush air through the instrument port. On the flipside of the card, with a gloved hand, hold the ScopeDryCheck™ in one of the corners with the blue side up ready to test if the lumen/channel is dry, while blowing air, aim the distal end of the instrument lumen or channel being flushed one (1)-cm away from the card. If any white dots are visible at any time during this test procedure, residual moisture was present in the lumen or channel. Repeat test until no white dots are visible.
Interpretation of Test Results	If white dots are visible on the card, residual moisture was present in the lumen or channel being flushed.
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	Prolonged exposure to UV light causes the indicator chemical to activate and fade over time.
Disposal	<ul style="list-style-type: none"> Single use only. Dispose of according to healthcare facility policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A

Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	<ul style="list-style-type: none"> • Store in a cool dry place at 60- to 80 °F with 30–60% rh. • Avoid exposure to moisture. • Avoid exposure to UV light.
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	N/A
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com