







Brand Name of Product	Handheld Labeling System
Generic Name of Product	Label Gun and labels
Product Code Number(s)	H-362026, H-369020, H369021, H369023, H369024, H369010, H369012, H369013, H369014, H369020S, H369021S, H369023S, H369024S, H369010S, H369012S, H369013S, H369014S.
Purpose of Product	The Handheld Labeling System has been designed specifically with Sterile Processing in mind for the labeling of any items requiring autoclaving (e.g., sterile linen packs, instrument trays, and peel pouches.)
Range of Applications for Product	Great versatility for labeling in other hospital departments such as dietary and pharmacy.
Key Specifications of Product	All labels are compatible with steam and ethylene oxide (EO) sterilization. The labels with an “S” at the end of the product code number (e.g., H369020S) are compatible with VH202 Sterilization.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	18–24 Months

Instructions for Using Product	
Description of Use(s)	Label Gun is used for labeling a variety of products and packages being autoclaved.
Preparation for Use	N/A
Diagrams (drawings, pictures)	<p>Setting the characters</p>   <p>1 2</p> <p>Labeling</p>    <p>1 2 3</p> <p>Replacing the label roll</p>    <p>1 2 3</p>    <p>3A 4 4A</p>

Replacing the ink roll		
		
1A	1B	2A
		
2B	3	

Steps for Use of Product	<p>Setting the characters</p> <ol style="list-style-type: none"> 1. Pull adjustment knob to desired position and turn knob to select character to print. 2. Push knob in before printing. (⚠ Important: failure to do so before operating could cause damage.) <p>Labeling</p> <ol style="list-style-type: none"> 1. Place your hand around the handle and squeeze your fingers upward to your palm. 2. Release the handle back to the starting position. 3. Pulling the handle will release the label from the opposite end of the label gun. <p>(Note: Repeat to use another label on another product or package.)</p> <p>Replacing the label roll</p> <ol style="list-style-type: none"> 1. Press and slide the red release for the top slide back. Release is on both sides of the label gun. 2. Lift the chamber up to insert the roll of labels. 3. Place new label roll inside with the label flap coming from underneath. 4. Secure the end of the label roll to the top end of the label gun. <p>(Note: When in use, the label will dispense through the chamber.)</p> <p>Replacing the ink roll</p> <p>(Precaution: We recommend wearing gloves to replace the ink roll. Your hands will get ink on them.)</p> <ol style="list-style-type: none"> 1. Lift the ink roll cover at the front of the label gun. 2. Remove the ink roller cartridge by sliding it out from the groove on either side. 3. Replace with new ink roller Prix by sliding cartridge back into the groove and close the cover.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • ⚠ CAUTION: Do <i>NOT</i> place the Handheld Label Gun in any sterilizer. • ⚠ CAUTION: Failure to push in the character adjustment knob before operating the Handheld Labeling System <i>could cause damage to the device.</i> • Recommended to wear gloves when changing the ink roll to prevent getting ink on your skin.
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	Use a damp cloth to wipe down the Handheld Label Gun.
Cleaning – Automated	N/A
Disinfection	N/A

Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	All the labels are compatible with Steam, VH202, and EO sterilization. Labels with an “S” at the end of the product code number are compatible with VH202 Sterilization. (⚠ DO NOT PLACE THE LABEL GUN IN ANY STERILIZER.)
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Premium Label Gun and two-part labels
Other Product Support Documents	Labeling Brochure, Labeling Price List
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge Company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com