

<b>Brand Name of Product</b>	HushKarts
<b>Generic Name of Product</b>	Transport carts
<b>Product Code Number(s)</b>	QCT-3301, QCT-225, QCT-2301-CG, QCT-235
<b>Intended Use</b>	Designed for quiet mobility. For use as surgical case carts and in CSD, Pharmacy, and Respiratory departments.
<b>Range of Applications for Product</b>	For use as surgical case carts and in the CSD, Pharmacy, and Respiratory departments.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• One (1)-piece, double-wall polyethylene.</li> <li>• Rubber cushioned 4-inch bearing casters.</li> <li>• Slip-resistant surface.</li> <li>• Raised sides on top.</li> <li>• HushKarts are washable.</li> <li>• Available colors: <ul style="list-style-type: none"> <li>○ Charcoal Gray</li> <li>○ Dark Brown</li> <li>○ Coffee Beige</li> <li>○ Dusty Rose</li> <li>○ Brick Red.</li> </ul> </li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	For use as surgical case carts and in CSD, Pharmacy, and Respiratory departments.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	N/A
<b>Steps for Use of Product</b>	<ul style="list-style-type: none"> <li>• Place “Clean” SST Trays inside for delivery to the OR and/or another clinical area.</li> <li>• Pick up “Dirty” containers at the same time and deliver to Decontam.</li> </ul>
<b>Interpretation of Test Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	Do not use a metal-bristle brush.
<b>Disposal</b>	N/A

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>• May be cleaned with a steam gun or scrubbed with a lint-free towel, sponge, or nylon-bristle brush. (NOTE: Never use a metal-bristle brush.)</li> <li>• Use a cleaning solution comprised of a non-abrasive, neutral pH detergent, and water (at the temperature recommended in the detergent manufacturer’s IFU).</li> <li>• Rinse thoroughly with Critical Water.</li> </ul>
<b>Cleaning – Automated</b>	<ul style="list-style-type: none"> <li>• Place HushKarts in the cart washer on cart wash cycle.</li> <li>• Consult and follow the washer manufacturer’s IFU.</li> <li>• Cycle Settings: <ol style="list-style-type: none"> <li>1. Two (2)-minute rinse with Critical Water from the facility’s cold water source.</li> <li>2. Detergent wash with neutral pH detergent. Follow the manufacturer’s IFU.</li> </ol> </li> </ul>

	3. Thermal disinfection rinse at 82 °C (180 °F) for 1-minute.
<b>Disinfection</b>	Use a facility approved wipe compatible with polyethylene.
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	SST Transport Carts
<b>Other Product Support Documents</b>	SST Brochure, SST Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com