

XEN

Chemistries
for the ultimate
peace of mind

Enzymatic



Orthopedics



Robotic
Instruments



Flexible
Scopes



Rigid
Scopes



Cart Washer



Manual



Ultrasonic



Washer/
Disinfectant

Instructions for Use

XEN Enzymatic is a pH-neutral, triple-enzyme liquid detergent for use in automated washer applications, including washer-disinfectors and cart washers, and manual washing. Enzymatic is low foaming to optimize performance of the automated washer. Its three enzymes—protease, lipase, and amylase—break down blood and proteins, fats and oils (lipids), and carbohydrates, respectively.

Order no.	Description
6036002601	Enzymatic 1 x 5 L.
6036002701	Enzymatic 1 x 5 gal.
6036002801	Enzymatic 1 x 15 gal.



Directions for Use in Automated Washing Equipment

1. The detergent should be automatically dosed into the wash cycle. Dosing is 0.06–1.02 oz./gal. (0.5–8.0 ml/l), depending on soil load and water quality. Higher doses will be required for higher soil loads and for poor water quality.
2. Automated washers should be set to dose at 95°F (35°C). The wash temperature should be set at 122°F–140°F (50°C–60°C).
2. If using an **Automated Dosing System** at the sink, for First Time Use with new product:
 - a. It is important that the product feed line is flushed with water before use in order to ensure the previous product is removed entirely.
 - b. Prime the line with Enzymatic to ensure that there is no/minimal air in the line.
 - c. Ensure the dosing is set to the indicated amount of 0.26–0.51 oz./gal. (2.0–4.0 ml/l).
3. Soak the items for a minimum of 2 minutes, ensuring sufficient time to obtain the desired cleaning. Cleaning times may be longer with a heavier soil load.
4. Rinse all items thoroughly after cleaning, ensuring any lumens are also flushed thoroughly.



Directions for Use in Manual Washing

1. Fill the sink or basin to an appropriate level with warm water (at a temperature of 77°F–113°F/25°C–45°C), ensuring the quantity of water used allows the correct dose to be calculated. Dosing is 0.26–0.51 oz./gal. (2.0–4.0 ml/l), depending on soil load and water quality. Higher doses will be required for higher soil loads and for poor water quality.



GETINGE 

Caution

- Do not mix with other chemicals.
- Wear correct PPE and follow applicable guidelines. Additional information on SDS.
- Check material compatibility before use.

Dispose of contents and container in accordance with local, regional, national, and international regulations.

Contains triethanolamine, protease, amylase, and lipase enzymes. For more ingredient information, visit www.getinge.com

Storage Information

Store in a cool, dry place protected from frost and away from acids and strong oxidizing agents. Store upright in original containers. Optimum storage temperature 41°F–77°F/5°C–25°C.

	Aluminum	Anodized Aluminum	Titanium	Copper	Brass	Gold	Nickel	Cobalt Chrome	Matte chrome plated	Stainless steel	Glass	Ceramics	Rubber	Plastic	Teflon™/ PTFE	Polycarbonate
Enzymatic	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•

- = The product is suitable for the material
- = Pay special attention to the guidelines from the instruments/device manufacturer

Teflon™ is a trademark of Chemours.

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2024 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Manufacturer: Quadralene Ltd, Unit 3, St. Modwen Park, Andressey Way, Derby, DE21 6YH, United Kingdom.
Sales Office, US: 1 Geoffrey Way, Wayne, NJ 07470, USA
Sales Office, Canada: 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada

www.getinge.com

