

Brand Name of Product	Angled Toothbrush Style Brush
Generic Name of Product	Toothbrush Style Brush
Product Code Number(s)	3184-10, 3184-100
Intended Use	To assist in cleaning the external surface of reusable instruments (e.g., distal end of flexible endoscopes and camera heads) with an angled-brush handle.
Range of Applications for Product	To assist in the cleaning of any flexible endoscope distal end and camera head.
Key Specifications of Product	<ul style="list-style-type: none"> • Single use only. • Nylon bristles • Polypropylene handle • Overall length: 16.40 cm.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	<ul style="list-style-type: none"> • Package of 10. • Box of 100.
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To assist in cleaning the external surface of the distal end of flexible endoscopes and camera heads.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Consult the manufacturer's IFU for the item to be cleaned. Follow the instructions closely. 2. Pay close attention to recommendations on the material composition of the brush trim (i.e., nylon, stainless steel). <p><i>(Note: This brush can also be used to clean the distal tip of flexible endoscopes and the opening of camera heads.)</i></p>
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Intended only for use as cleaning tools. They are not for use in patient care or procedure areas. • Do not reuse or reprocess Angled Toothbrush style brushes. • Excessive force or use of a stiff brush when brushing may: <ul style="list-style-type: none"> ○ Scratch the distal end. ○ Result in water leakage. ○ Cause the elevator wire to come off the distal end. ○ Bend or kink the elevator wire so that the forceps elevator will no longer work. • ⚠ Used Angled Toothbrushes are to be treated as biohazardous.
Disposal	<ul style="list-style-type: none"> • Single use only. • Treat used/finished Angled Toothbrushes as biohazardous. Dispose of in accordance with your facility's guidelines (in the biohazard container).

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A

Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	<ul style="list-style-type: none"> • Check the brush head. <ul style="list-style-type: none"> ○ Examine the bristles for any damage, loose, or missing bristles. ○ If the bristles are crushed, gently straighten them with your gloved hands. ○ Visually check for debris on the bristles of the brush head. If there is debris on the brush: <ul style="list-style-type: none"> ▪ Immerse the brush in water. ▪ Clean the brush until no further debris is observed on the brush. • Check the handle for scratches and other damage. • Check the grip section for scratches and other damage.
Reassembly Instructions	N/A
Packaging	<ul style="list-style-type: none"> • Package of 10. • Box of 100.
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	Endoscope Brushes IFU, Elevator Mechanism IFU
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com