

Getinge Assured

SafeStep - ATP Monitor

The Getinge Assured ATP Monitor detects the presence of ATP in just 15 seconds. By detecting ATP (adenosine triphosphate) – the energy molecule found in all human, animal, plant, bacterial, yeast, and mold cells – SafeStep gives you the testing power to identify contaminants quickly.



Recommended Pass/Fail Criteria

Application	Pass (RLU)	Fail (RLU)
Instrument Surfaces and Channels	0 to 100	101 and over
Sterile Processing - General (all non-critical surfaces in procedure rooms, restrooms, waiting rooms, etc. for testing counters, bedrails, blood pressure cuffs, toilets, faucets, hand rails, beds, computers, I.V. poles, etc.)	0 to 45	46 and over

Ordering Information

Getinge Assured	Order Number
SafeStep ATP Monitor	61301606628
SafeStep Test Swabs (100/box)	61301606629
SafeStep 1mm Test Sponges (100/box)	61301606630
SafeStep 1.5mm Test Sponges (100/box)	61301606631
SafeStep 2mm Test Sponges (100/box)	61301606632
SafeStep 3mm Test Sponges (100/box)	61301606633
SafeStep 4mm Test Sponges (100/box)	61301606634
SafeStep 5mm Test Sponges (100/box)	61301606635
SafeStep Test Sponge Sample Pack (45/bag)	61301606693



SafeStep Handheld Unit

- Quick results - just 15 seconds
- Stores up to 2,000 test results
- V2 Software allows up to 5,000 (Previously 250) Programmable names with Pass/Fail thresholds and 200 (Previously 19) Users

SafeStep Software

- Generate reports and graphs to analyze data and trends
- Identify trends - allowing you to take corrective action
- Data allows you to evaluate established workplace hygiene protocols



SafeStep Test Swab

- Informs the user of absence or presence of residual soil or contamination on the surface of instruments, examination rooms, operating rooms, restrooms, waiting rooms and Health Care Practitioner hands
- User-friendly, all-in-one sampling test
- Premoistened swab for easy maneuverability
- Write-on swab label
- Unprecedented accuracy and reproducibility
- 12-month shelf life at refrigerated temperatures (36°F to 47°F) (2°C to 8°C)
- 4 week shelf life at room temperatures (70°F) (21°C)



SafeStep Test Sponges

- Provides an accurate numerical reading of the level of contaminants present inside instrument channels after cleaning
- Simple, quick and user friendly
- Absorbent swab
- Available in various sizes to meet the channel diameter of your instruments ranging from 1mm-5mm
- Flexible wand that allows for easy maneuvering through complex internal channels
- Color-coded package for easy selection of correct size

With a firm belief that every person and community should have access to the best possible care, Getinge provides hospitals and life science institutions with products and solutions aiming to improve clinical results and optimize workflows. The offering includes products and solutions for intensive care, cardiovascular procedures, operating rooms, sterile reprocessing and life science. Getinge employs over 10,000 people worldwide and the products are sold in more than 135 countries.

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. • Copyright 2019 Getinge AB or its subsidiaries or affiliates. • All rights reserved. • MCV00095087 REVA

Manufactured by Ruhof

Sales Office: Getinge USA Sales, LLC 45 Barbour Pond Drive, Wayne, New Jersey 07470

Sales Office, Canada: 90 Matheson Blvd West, Suite 300 • Mississauga, Ontario L5R 3R3 • Canada

www.getinge.com

