



<b>Brand Name of Product</b>	Tray Labels
<b>Generic Name of Product</b>	Tray Labels
<b>Product Code Number(s)</b>	VND-TL, LN-TL
<b>Intended Use</b>	Tray Labels facilitate specific markings/remarks for trays. Designed to minimize misunderstandings about an instrument tray's contents, and to notify about the maintenance of an item.
<b>Range of Applications for Product</b>	N/A
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Labels attach with cable tie onto containers</li> <li>• Cable ties and a sterilizable marker are included.</li> <li>• Compatible with steam, ethylene oxide (EO), and Vaporized hydrogen peroxide (VHP) sterilization.</li> <li>• Available in a variety of colors.</li> <li>• Single Use</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	Packaged: 500 Tray Labels per box; 500 cable ties; and one (1)-marker
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	The polyethylene terephthalate (PET) based label is suitable for sterilization and is resistant to washing and disinfection.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	  <p style="text-align: center;"><b>Figure 1</b> <span style="margin-left: 200px;"><b>Figure 2</b></span></p>
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Fill in the information on the designated lines for the Loaner label (i.e., Loaned From, Loaned To, By, and Date). <b>(Fig.1)</b>.</li> <li>2. Fill in the information on the designated line for the Vendor label (i.e., Company Name, Tray Name, and Date of Surgery). <b>(Fig.2)</b>.</li> </ol>
<b>Interpretation of Test Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Do not allow labels to be in physical contact with surgical instruments during sterilization cycles.</li> <li>• Do not block any vent holes in trays with labels, as this may interfere with sterilization.</li> <li>• <b>⚠ Attention: HOT!</b> Do not handle labels until they cool to room temperature after sterilization or thermal disinfection.</li> </ul>
<b>Disposal</b>	Labels should be disposed of in accordance with facility policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A

<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	Compatible with washer disinfectors.
<b>Disinfection</b>	The thermal disinfection stage of an automated washer is sufficient for disinfection of the label.
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Compatible with steam, EO, and VHP sterilization.</li> <li>• Recommended cycle time for steam sterilization: <ul style="list-style-type: none"> <li>○ 132 °C (270 °F) for 4 minutes</li> <li>○ 135 °C (275 °F) for 3 minutes.</li> </ul> </li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	Make sure the label is hanging freely from the tray handle and not blocking holes in the tray.
<b>Related Healthmark Products</b>	Self-Looping Tray Labels
<b>Other Product Support Documents</b>	Labels Brochure, Labels Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>