

Getinge Assured

ChemiSix Sterilization Emulator

Directions for Use

1. Place a ChemiSix CI in each pack, peel pouch or tray for sterilization.
2. Sterilize the load as usual in a steam sterilizer.
3. At the completion of the sterilization cycle verify that indicator arrow has turned completely to green.
NOTE: Pack, pouch or tray should only be opened at the time of use as items are considered unsterile once opened. Follow hospital policy for load release.
4. If the indicator arrow is not completely turned to green, it suggests inadequate sterilization conditions and items must be re-sterilized (per hospital policy) with new/unprocessed indicator strips.

Caution

- If there is any doubt about the sterility of item, it must be considered NOT sterile.
- Do not use any indicator strips with bars that are not purple in color prior to exposure to steam sterilization processing.
- Getinge ChemiSix CIs are not recommended for use in gravity cycles.
- Expiry date is printed on product packaging. Use of indicators beyond the expiration date is not recommended.

Storage

Store between 50°F - 100°F (10°C - 38°C) at normal humidity. Keep unexposed strips in the re-sealable bag provided.

Standards Conformity

ISO 11140-1, Class 6

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries.
Copyright 2020 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Sales Office: Getinge USA Sales, LLC 45 Barbour Pond Drive, Wayne, New Jersey 07470
Sales Office, Canada: 90 Matheson Blvd West, Suite 300 • Mississauga, Ontario L5R 3R3 • Canada

www.getinge.com

DF-155 REVD